+43°C

## Vaccine and Icepack Freezer MF114, 214, 314

- GB Instructions for use
- FR Consignes d'utilisation
- ES Instrucciones para el uso
- RUS Инструкция по эксплуатации
- CHI 使用说明

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# Warning

**WARNING:** Keep ventilation openings in the appliance's cabinet or in the built-in structure clear of obstruction

**WARNING:** Do not use other **mechanical devices** or other means to accelerate the defrosting process than those recommended by the manufacturer

**WARNING:** Do not damage the refrigerant system

**WARNING:** Do not use **electrical appliances** inside the refrigerated storage compartment, unless they are of a type recommended by the manufacturer

**WARNING:** Do not expose the appliance to rain

**WARNING:** This appliance is not intended for use by young children or infirm persons unless they have been adequately supervised by a responsible person to ensure that they can use the appliance safely. Young children should be supervised to ensure that they do not play with the appliance

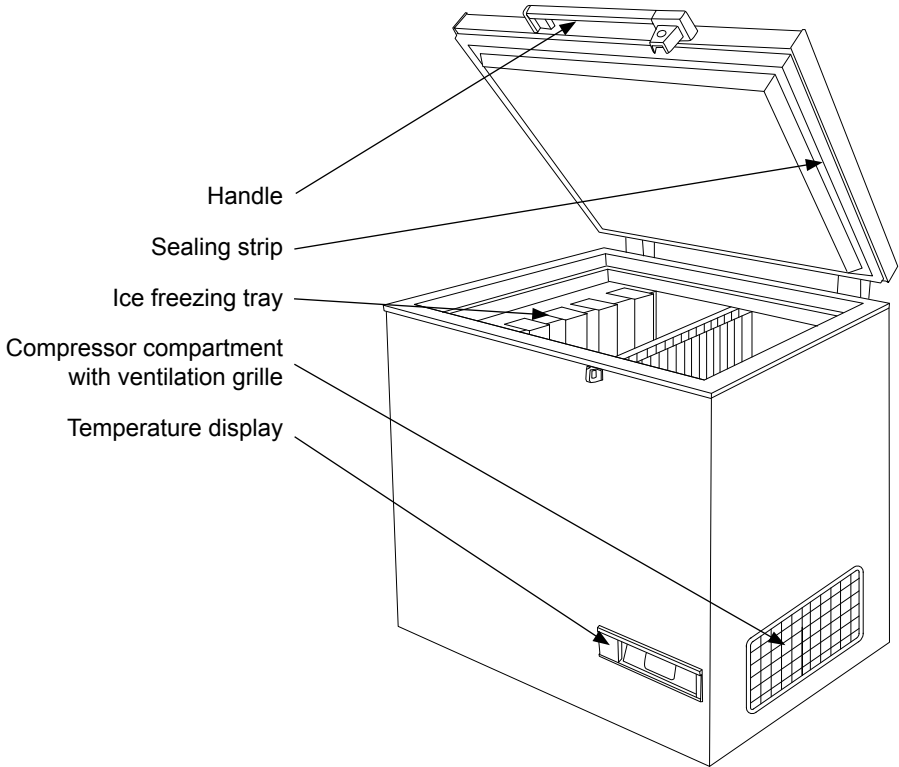
- Always keep the keys in a separate place and out of reach of children
- Before servicing or cleaning the appliance, unplug the appliance from the mains or disconnect the electrical power supply
- If the supply cord is damaged, it must be replaced by the manufacturer, its service agent, or similarly qualified persons in order to avoid a hazard
- Frost formation on the interior evaporator wall and upper parts is a natural phenomenon. Therefore, the appliance should be defrosted during normal cleaning or maintenance.
- Please note that changes to the appliance construction will cancel all warranty and product liability

# Contents

Warning .....	2
Get to know your freezer .....	4
General information .....	5
Power supply .....	5
Technical data .....	5
Unpacking and installation .....	5
Preparation .....	6
Temperature adjustment .....	6
Cool down of the appliance .....	6
Loading the appliance .....	7
Defrosting and cleaning .....	7
Trouble shooting .....	8
Warranty, Spare parts and service .....	9
Wiring diagram .....	16
Disposal .....	17

Reserving the right to alter specification  
without prior notice.

# Get to know your Freezer

## General information

MF-Model is a vaccine storage freezer and icepack freezer. It has two compartments; one for vaccine storage and one for icepack freezing.

### Power supply

The MF vaccine freezer is equipped with a heavy-duty compressor that can operate at voltages 22% below nominal voltage. See the rating plate. If voltage variations go beyond this range it is recommended to use an automatic voltage stabilizer.

### Technical data

This device complies with relevant EU directives including Low Voltage Directive 2006/95 EEC. and Electromagnetic Compatibility Directive 2004/108/EC

The rating plate provides various technical information as well as type and serial number.

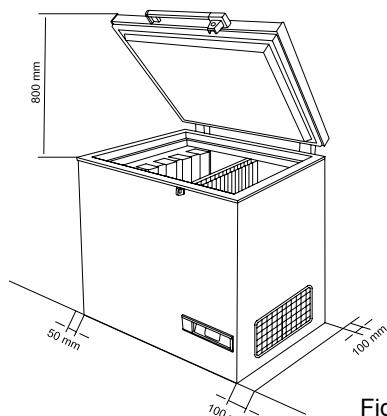
Fig. 2

**IMPORTANT!** There must be free access to the ventilation grille

## Unpacking and installation

### Unpacking

1. Unpack the appliance and check that it has not been damaged. If you observe any damage, inform your supervisor.
2. Check the rating for correct voltage and frequency.
3. Open and remove all internal packing material.

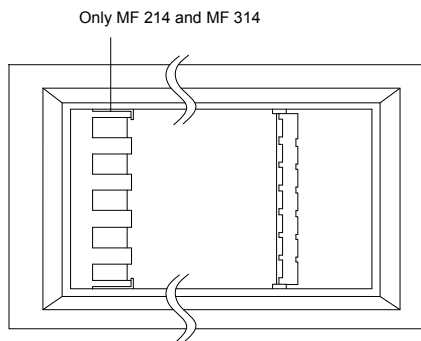
Fig. 1

### Installation

The appliance must be placed in a well ventilated room, not in direct sunlight and away from other heating sources.

Place the appliance on the floor, ensure that it is level. Distance between the wall and the back and left side of the appliance must be minimum 50 mm and the distance between the wall and the right side of the appliance must be minimum 100 mm. (fig. 2).

If installed in a room with other refrigerators/freezers, ensure a minimum distance between the appliances of minimum 200 mm. There must be at least 30 mm clearance between the base of the appliance and the floor.

# Preparation

Check that the plug fits your type of socket. If it does not fit, have a qualified electrician to fit the plug. This appliance must be earthed.

Lead of cord		Pin of plug
230V	115V	
Brown	White	Live
Blue	Black	Neutral
Green/Yellow	Green	Earth

Connect the plug to the socket and switch on the appliance.

Check, by listening, that the compressor is running, if it does not run, contact your supervisor.

# Cool down of the appliance

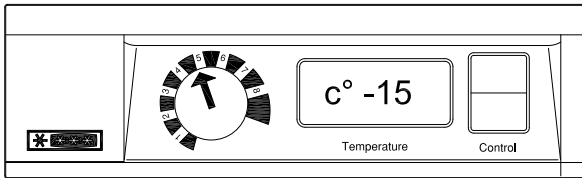
Fig. 4

Before the appliance is loaded with vaccines ensure that the temperature is below -15 °C:

- Set the thermostat at position 5
- Check the temperature on the thermometer

# Temperature adjustment

This appliance is equipped with one thermostat. See Fig. 3 and Fig. 4

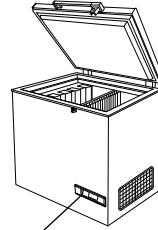
Fig. 3

The thermostat is a electronic thermostat that controls the temperature in the vaccine and icepack freezing compartments. Setting 1 applies minimum cold and setting 8 maximum cold.

- If the temperature in the vaccine compartment is above -15°C the thermostat should be adjusted. The temperature in the vaccine compartment must always be monitored on the thermometer and be within the range -15 to -20°C.

Due to the mechanical tolerance of the thermostats you should always control the temperature during cool down .

## Loading the appliance

### Loading vaccines

When the temperature in the freezer has stabilized, i.e. the temperature should be below  $-15$  to  $-20^{\circ}\text{C}$  and the compressor stops and starts, vaccines can be loaded. The vaccines should be placed in the mid part.

### Loading and freezing of icepacks

The icepacks should be placed in the ice-pack compartment and the suspended tray (MF 214 and MF 314). Make sure you have a good contact between the icepack and the evaporator surface. Fig. 5

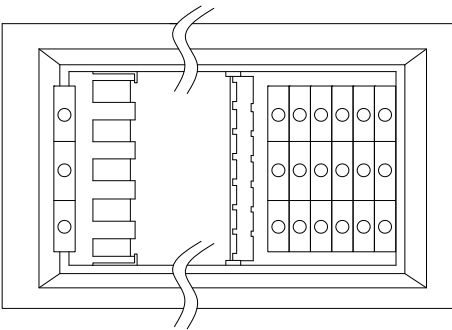
Fig. 5

**If thermostat is set to a warmer position, remember to check the temperature in the vaccine compartment**

## Defrosting and cleaning

For the appliance to operate well and to save energy, it is important that it is cleaned and defrosted regularly.

When opening the appliance, moisture enters and settles on inner cold surfaces and forms a layer of frost or ice.

The layer of ice makes the freezer less efficient and must be removed, "defrosted".

It is recommended that the appliance be defrosted every third month.

### Defrosting

- Before defrosting the vaccines must be moved to another working freezer or cold box with icepacks. Switch off the appliance and pull out the plug.
- Open the lids and leave it open.
- Keep the lids open to allow the frost to melt completely.

### Cleaning

- Once the appliance is defrosted it must be cleaned.
- Clean the inside of the appliance with a mixture of lukewarm water and a mild detergent.
- Allow the cleaned parts to dry completely.
- Clean the outside of the appliance with a mixture of lukewarm water and a mild detergent or soap.
- Close the lid and follow the procedures in "Cool down of the appliance" page 6

## Trouble shooting

<b>Fault</b>	<b>Possible cause</b>	<b>Remedy</b>
Compressor is not running, and the ice packs are not cold	Be patient, it is most likely that the compressor will start within a few minutes.	If this is not the case, check the following: <ul style="list-style-type: none"><li>- Check that power is connected.</li><li>- Check the fuse and replace it if necessary.</li><li>- If the above is OK, call technical supervisor.</li></ul>
Compressor is running, and the temperature is too high	The ventilation grille is blocked.  The lid is not closed properly.  The temperature in the room in which the appliance is installed is too high.	Ensure unhindered air circulation.  Ensure that the lid is closed properly.  Shield the appliance against direct sun light and ensure more ventilation to the room.
No temperature is displayed	The thermometer is broken.  There is not enough light for the solar sensor.	Change the thermometer.  Turn on the light.

## Warranty, spare parts and service.

### Warranty disclaimer

Faults and damage caused directly or indirectly by incorrect operation, misuse, insufficient maintenance, incorrect building, installation or mains connections. Fire, accident, lightning, voltage variation or other electrical interference, including defective fuses or faults in mains installations are not covered by the warranty.

Repairs performed by others than approved service centres and any other faults and damage that the manufacturer can substantiate are caused by reasons other than manufacturing or material faults are not covered by the warranty.

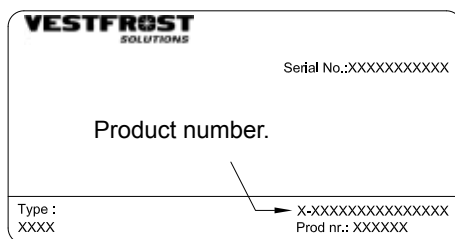
Please note that changes to the construction of the appliance or changes to the component equipment of the appliance will invalidate warranty and product liability, and the appliance cannot be used lawfully. The approval stated on rating plate will also be invalidated.

Transport damage discovered by the buyer is primarily a matter to be settled between the buyer and the distributor, i.e. the distributor must ensure that such complaints are resolved to the buyer's satisfaction.

Before calling for technical assistance, please check whether you are able to rectify the fault yourself. If your request for assistance is unwarranted, e.g. if the appliance has failed as a result of a blown fuse or incorrect operation, you will be charged the costs incurred by your call for technical assistance.

### Spare parts

When ordering spare parts, please state the type, serial and product numbers of your appliance. This information is given on the rating plate. The rating plate contains various technical information, including type and serial numbers.


## MF 114

---

<u>Position</u>	<u>Item number</u>	<u>Item name</u>
0061	0-6538001	Filter drier, 20 g
0070	A921001	Cover + wirring clamp
0071	A921115	Starting device, 103N0021
0087	0-6038175	Base plate fittings, set, complete
0126	8-036038325	Compressor FR8,5G OC
0898	0-7211046	Phail, air
3125	3010457	Pipe holder Ø6
5000	3010049	Drain plug
5002	6000959	Foot with pin
5057	6010084	Condenser
5108	5000917670	Lid foamed without handle/hinges
5109	3020102-01	Rubber gasket
5375	6000965	Partition, complete
5401	1510028	Hinge without spring
5402	1510029	Hinge 40
5450	3010040-01	Top part for hinge cover
5451	3010032-01	Bottom part for hinge cover
5605	304090501	Handle with lock (push and turn)
5606	8090341	Inlay for handle, neutral
5682	1510046	Key, set - 2pc.
5683	2040145	Catch for handle
5684	3010265-01	Cover for catch
5685	600098801	Lock with keys,(push and turn)
5710	7020239	Thermostat, freezer
5714	7020134	Thermostat phial (Fan box )
5738	7010027	Pilot lamp, green
5756	7020164	Thermometer, solar
5771	3010051-01	Thermostat knob
5783	3040146	Control panel
5870	6000902	Fan
5901	353090001	Basket with handle, white
5902	3010107-01	Spare handle for basket
5970	3010308-01	Motor screen

### Please Note!

The drawing covers all variants of the model and therefore contains more pos. numbers than your product. See the spare parts list for exact product specification.

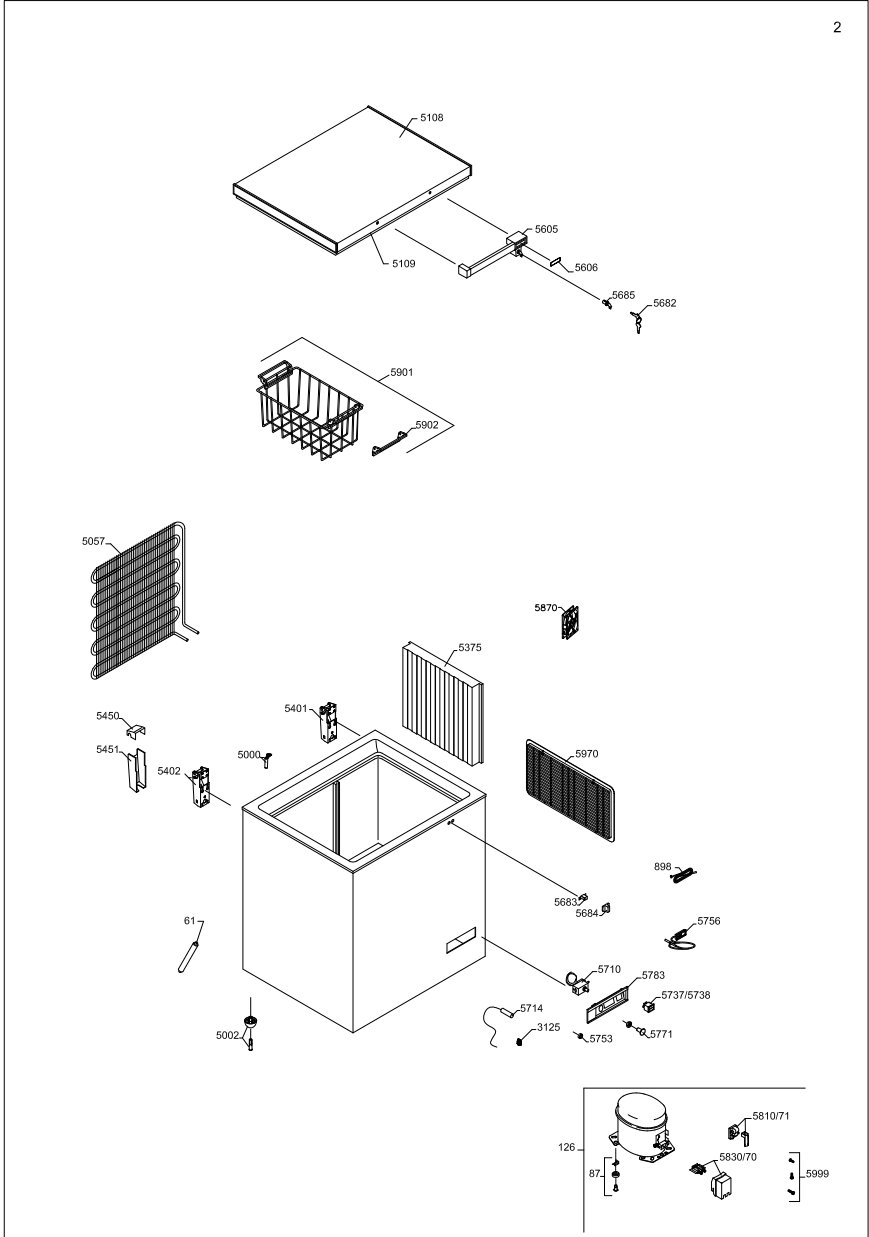
## MF 214

Position	Item number	Item name
0061	0-6538001	Filter drier, 20 g
0070	A921001	Cover + wirring clamp
0071	A921115	Starting device, 103N0021
0087	0-6038175	Base plate fittings, set, complete
0126	8-036038325	Compressor FR8,5G OC
0898	0-7211046	Phail, air
3125	3010457	Pipe holder Ø6
5000	3010049	Drain plug
5002	6000959	Foot with pin
5057	6010084	Condenser
5116	5000918651	Lid foamed without handle/hinges
5117	3020106-01	Rubber gasket
5374	6000966	Ice pack tray
5375	6000965	Partition, complete
5401	1510031	Hinge 30
5450	3010040-01	Top part for hinge cover
5451	3010032-01	Bottom part for hinge cover
5605	304090501	Handle with lock (push and turn)
5606	8090341	Inlay for handle, neutral
5682	1510046	Key, set - 2pc.
5683	2040145	Catch for handle
5684	3010265-01	Cover for catch
5685	600098801	Lock with keys,(push and turn)
5710	7020239	Thermostat, freezer
5714	7020134	Thermostat phial (Fan box )
5738	7010027	Pilot lamp, green
5756	7020164	Thermometer, solar
5771	3010051-01	Thermostat knob
5783	3040146	Control panel
5870	6000902	Fan
5901	353090001	Basket with handle, white
5902	3010107-01	Spare handle for basket
5970	3010308-01	Motor screen

**Please Note!**

The drawing covers all variants of the model and therefore contains more pos. numbers than your product. See the spare parts list for exact product specification.

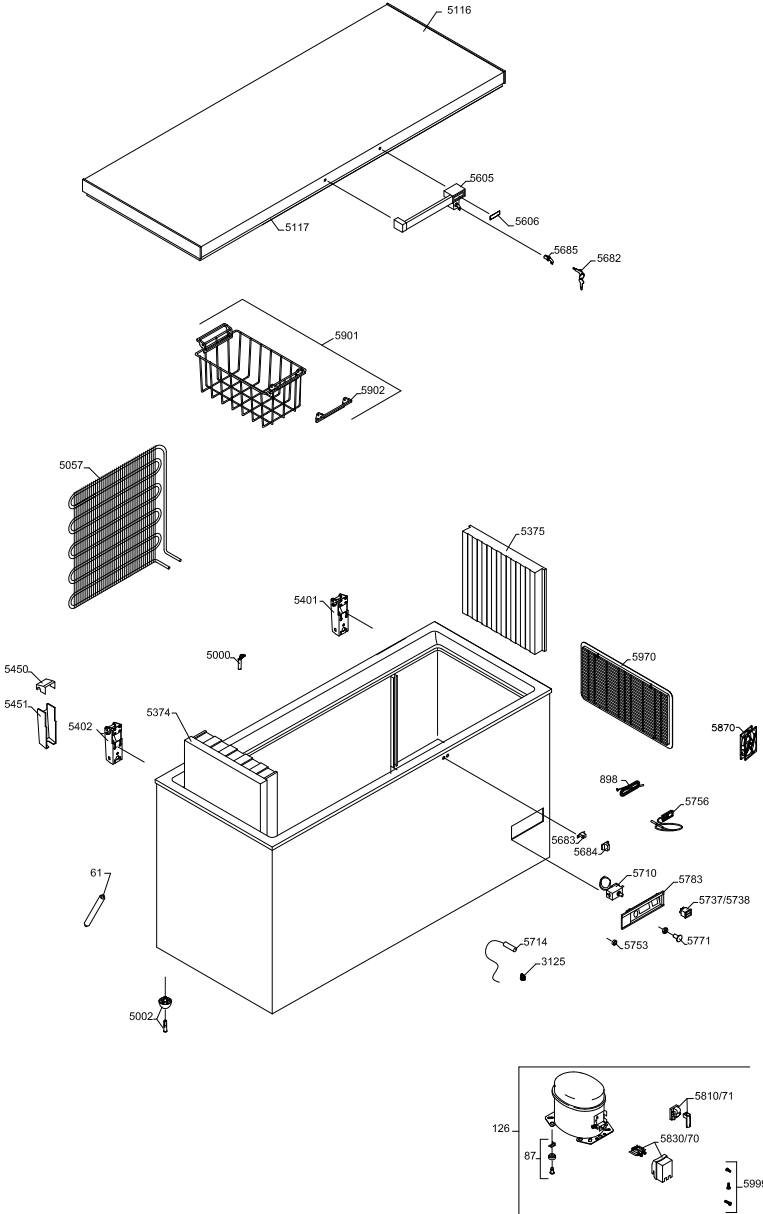
## MF 314

---

Position	Item number	Item name
0061	0-6538001	Filter drier, 20 g
0070	A921001	Cover + wiring clamp
0071	A921115	Starting device, 103N0021
0087	0-6038175	Base plate fittings, set, complete
0126	8-036038858	Compressor - FR 10G, Freezer, Complete, R134a , OK
0898	0-7211046	Phail, air
3125	3010457	Pipe holder Ø6
5000	3010049	Drain plug
5002	6000959	Foot with pin
5057	6010084	Condenser
5122	5000914651	Lid foamed without handle/hinges
5123	3020109-01	Rubber gasket
5374	6000966	Ice pack tray
5375	6000965	Partition, complete
5402	1510029	Hinge 40
5450	3010040-01	Top part for hinge cover
5451	3010032-01	Bottom part for hinge cover
5605	304090501	Handle with lock (push and turn)
5606	8090341	Inlay for handle, neutral
5682	1510046	Key, set - 2pc.
5683	2040145	Catch for handle
5684	3010265-01	Cover for catch
5685	600098801	Lock with keys,(push and turn)
5710	7020239	Thermostat, freezer
5714	7020134	Thermostat phial (Fan box )
5738	7010027	Pilot lamp, green
5756	7020164	Thermometer, solar
5771	3010051-01	Thermostat knob
5783	3040146	Control panel
5870	6000902	Fan
5901	353090001	Basket with handle, white
5902	3010107-01	Spare handle for basket
5970	3010308-01	Motor screen


### Please Note!

The drawing covers all variants of the model and therefore contains more pos. numbers than your product. See the spare parts list for exact product specification.
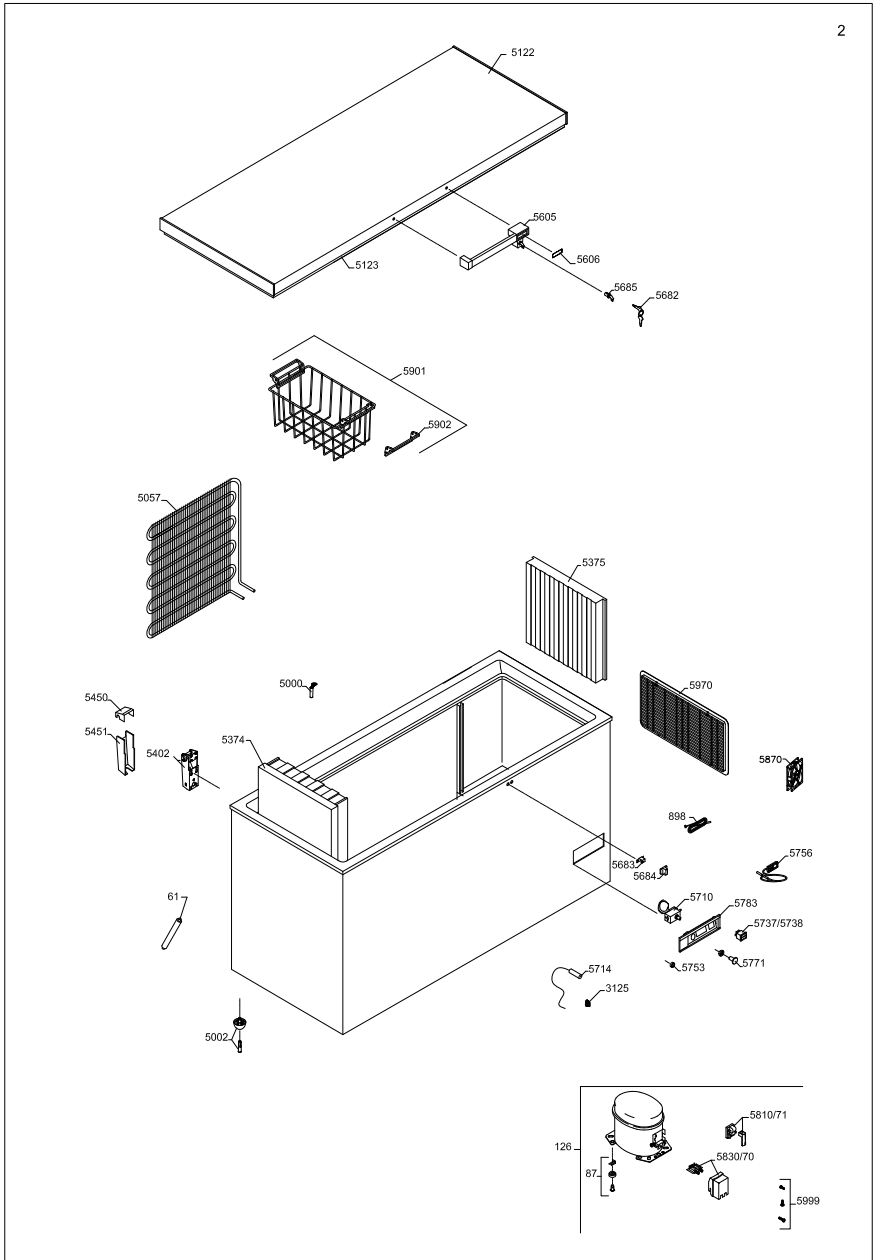
# Wire-diagram

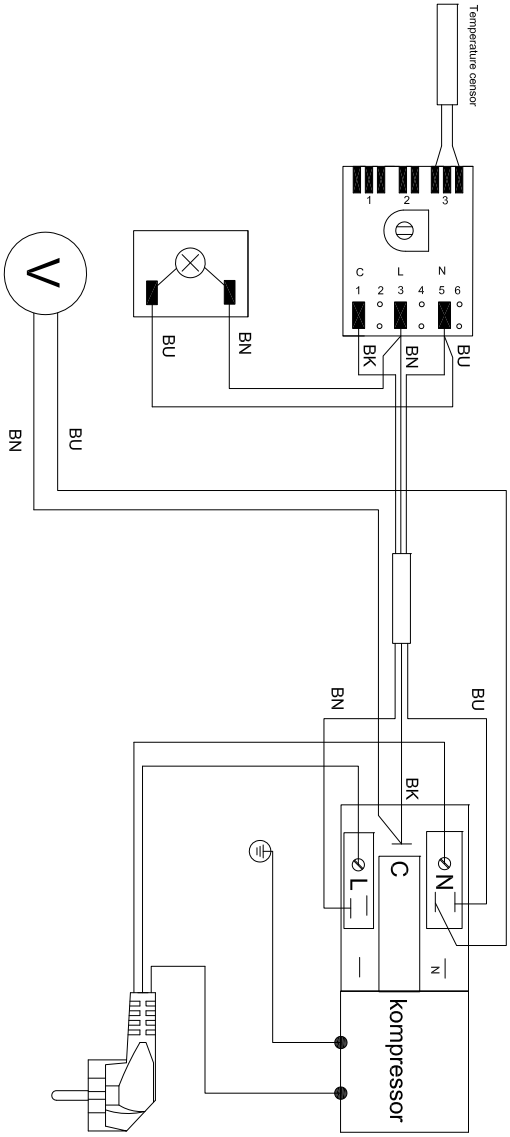1006196-01

-	Farnkodelle	Colour-Code	Farnkodelle
BN	Braun	Brown	Braun
BU	Blau	Blue	Blau
BK	Schwarz	Black	Schwarz
LI	Lila	Lilac	Lila
VG	Gul / groen	Yellow / green	Gul / groen
VH	Wit	White	Wit
RD	Rood	Red	Rood

# Disposal

## *Information for Users on Collection and Disposal of Old Equipment and used Batteries*

These symbols on the products, packaging, and/or accompanying documents mean that used electrical and electronic products and batteries should not be mixed with general household waste. For proper treatment, recovery and recycling of old products and used batteries, please take them to applicable collection points, in accordance with your national legislation and the Directives 2002/96/EC and 2006/66/EC.

By disposing of these products and batteries correctly, you will help to save valuable resources and prevent any potential negative effects on human health and the environment which could otherwise arise from inappropriate waste handling.

For more information about collection and recycling of old products and batteries, please contact your local municipality, your waste disposal service or the point of sale where you purchased the items.

Penalties may be applicable for incorrect disposal of this waste, in accordance with national legislation.

## **For business users in the European Union.**

If you wish to discard electrical and electronic equipment, please contact your dealer or supplier for further information.

## **[Information on Disposal in other Countries outside the European Union]**

These symbols are only valid in the European Union. If you wish to discard this product, please contact your local authorities or dealer and ask for the correct method of disposal.

### **Note for the battery symbol:**

This symbol might be used in combination with a chemical symbol. In this case it complies with the requirement set by the Directive for the chemical involved.

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